

# RENAL BIOPSY KIT


## STORE REFRIGERATED / SHIP REFRIGERATED



REFRIGERATE (4°C) KIT IMMEDIATELY UPON ARRIVAL



1. Place the specimen containers into the refrigerated specimen bag.

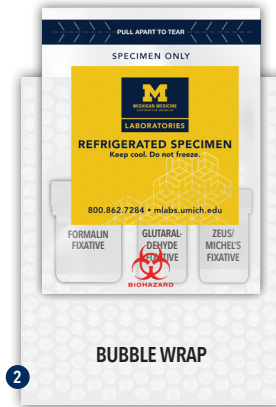

2. Secure specimen bag in bubble wrap.



3. Place refrigerated cold pack into the bottom of the box along with wrapped specimen bag.
4. Completed requisition and clinical history documentation should be placed into the sleeve on top of the Styrofoam container.
5. Secure box lid with rubber band. Transport specimens in this box for tracking.