

FRESH MUSCLE BIOPSY KIT

STORE REFRIGERATED / SHIP REFRIGERATED



REFRIGERATE (4°C) GLUTARALDEHYDE / COLD PACK IMMEDIATELY UPON ARRIVAL



SPECIMENS MUST ARRIVE AT MLABS WITHIN **4 HOURS** OF COLLECTION.



Place the specimen containers into the refrigerated specimen bag.



Secure specimen bag in bubble wrap.



Place refrigerated cold pack into box along with wrapped specimen bag, completed requisition and clinical history documentation. Transport specimens in this box for tracking.