

HEMATOPATHOLOGY CONSULT KIT

Bone Marrow


STORE AND SHIP AT ROOM TEMPERATURE



Specimens should arrive at MLabs within **12 HOURS (optimal results)** and not exceeding 24 hours from collection



To avoid formalin contamination of slides, package the formalin vials and slides/vacutainer tubes in two separate room temperature specimen bags.

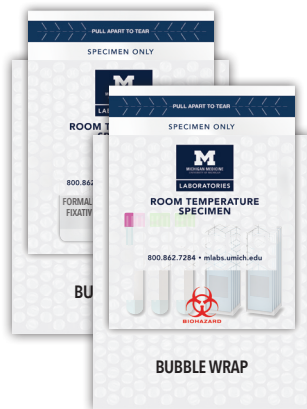

Secure each specimen bag in a separate bubble wrap pouch.



Place bubble wrap pouches in box along with completed requisition. Include a copy of the patient's most recent complete blood cell and platelet count (CBC) (preferred), or additional lavender top tube (EDTA) containing peripheral blood if CBC is not available. Transport specimen in this box for tracking.