


# STEP 3: BOX B

## FROZEN MUSCLE BIOPSY KIT



**REFRIGERATE (4°C) GLUTARALDEHYDE / COLD PACK IMMEDIATELY UPON ARRIVAL**

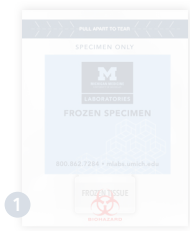

### PACK FROZEN SPECIMEN

1. **STEP 1:** Refer to protocol: Specimen handling, frozen tissue, step 1.
2. Set frozen specimen bag on top of 3.5lbs of dry ice
  - Fill with additional 3.5lbs of dry ice on top of the frozen specimen bag for a total of 7lbs.
  - Place the lid onto the Styrofoam Box A, and secure with rubberband.



### PACK REFRIGERATED SPECIMEN

3. Place the formalin fixative and glutaraldehyde fixative containers into the refrigerated specimen bag and seal.
4. Secure specimen bag in bubble wrap.
5. Place the cold pack at the bottom of **Box B**. Set the bubble-wrapped specimen bag on top, add extra bubble wrap, then close and secure the lid with a rubber band.



### PACK SHIPPING CONTAINER

6. Place Styrofoam Box A at bottom of shipping box.
7. Stack Styrofoam **Box B** on top of Box A.
8. Place the requisition, and clinical history **ON TOP** of Styrofoam Box B.
9. Transport in shipping container provided for specimen tracking.